

















# Clorox Healthcare® Equipment Compatibility Guide

		Bleach				Hydrogen Peroxide	Quat	
<b>Beds/Pads</b>								
		Clorox Healthcare® Bleach Germicidal Wipes	Clorox Healthcare® Bleach Germicidal Cleaners	Clorox Healthcare® Fuzion Cleaner Disinfectant	Dispatch® Hospital Cleaner Disinfectant Towels with Bleach	Clorox® Germicidal Bleach	Clorox Healthcare® Hydrogen Peroxide Cleaner Disinfectant Wipes	Clorox Healthcare® VersaSure Alcohol-Free Disinfectant Wipes
Manufacturer	Product							
MIDMARK	Medical Exam Bed (Acute & Physicians)							
MIZUHOSI	Surgical Pads							
SKYTRON®	Surgical Compressions Pad, and Surgical Table Disinfecting							
	Hercules 6701 Surgical Table							
STRYKER	Beds							
	Stretchers							
	Patient Support Surfaces							
	Furniture							
	EMS cots							
	Fasteners							
	Model D860 Operating Table							
Hill-Rom	Mattresses *Occasional use only; rinse with tap water after use							
	Envella Air Fluidized Therapy System							
AliMed	AliGel O.R. Products Pads and Positioners					6%		
Curbell Medical	Universal Mattress							
Anacom	MedTek Series 7 and 8 Pillow Speaker						Clorox Disinfecting Wipes	
Medical Positioning, Inc.	UltraScan Table/GSPM Table/ Rapid Response HUT Table/ HUT Table							
Bella VMS	Waterproof Vet Bed/ Mat for Fold Down Exam Tables							



**Legend**

-  Product is mentioned by name in Cleaning & Care guide and is approved for use; if relevant, dilution is stated
-  Active ingredient/level is listed in Cleaning & Care guide as approved for use; if relevant, dilution is stated








# Clorox Healthcare® Equipment Compatibility Guide

		Bleach				Hydrogen Peroxide	Quat	
<b>Blood Glucose Meters</b>		 Clorox Healthcare® Bleach Germicidal Wipes	 Clorox Healthcare® Bleach Germicidal Cleaners	 Clorox Healthcare® Fuzion Cleaner Disinfectant	 Dispatch® Hospital Cleaner Disinfectant Towels with Bleach	 Clorox® Germicidal Bleach	 Clorox Healthcare® Hydrogen Peroxide Cleaner Disinfectant Wipes	 Clorox Healthcare® VersaSure Alcohol-Free Disinfectant Wipes
Manufacturer	Product							
Infab	FreeStyle InsuLinx®							
	FreeStyle Lite®, Freedom Lite®							
	i-STAT® System							
	Precision Xtra™ Glucose & Ketone							
ARKRAY	ASSURE® 4							
	ASSURE® Platinum Glucose Meter							
	ASSURE® Prism							
	ASSURE® Pro							
Bayer	CONTOUR® Link							
	CONTOUR® USB Meters							
EPS Biotech	EasyMax® Glucose Meter							
Lifescan	OneTouch® Blood Glucose Meters							
Nova Biomedical	Nova StatStrip/StatSensor/Stat Xpress Glucose Hospital Meter							
Roche Diagnostics	ACCU-CHEK® Advantage Meter System							
	ACCU-CHEK® Inform Blood Glucose Monitor							
	ACCU-CHEK® Inform II Base Unit							
Medline	EVENCARE G2							
	EVENCARE G3							
Abbott	FreeStyle Lite and InsuLinx Blood Glucose Monitoring System							

**Legend**

-  Product is mentioned by name in Cleaning & Care guide and is approved for use; if relevant, dilution is stated
-  Active ingredient/level is listed in Cleaning & Care guide as approved for use; if relevant, dilution is stated

# Clorox Healthcare® Equipment Compatibility Guide

		Bleach				Hydrogen Peroxide	Quat	
<b>Infusion Pumps</b>								
		Clorox Healthcare® Bleach Germicidal Wipes	Clorox Healthcare® Bleach Germicidal Cleaners	Clorox Healthcare® Fuzion Cleaner Disinfectant	Dispatch® Hospital Cleaner Disinfectant Towels with Bleach	Clorox® Germicidal Bleach	Clorox Healthcare® Hydrogen Peroxide Cleaner Disinfectant Wipes	Clorox Healthcare® VersaSure Alcohol-Free Disinfectant Wipes
Manufacturer	Product							
BD ALARIS PUMPS (FORMLY CAREFUSION)	CareFusion Alaris® Systems							
HOSPIRA	Omni-Flow 4000® Plus IV Medication Management System							
	GemStar® Infusion Pumps							
AGILIA INTUITIVE GENERATION	Volumetric Infusion Pump							
BAXTER	ASSURE® 4							








**Legend**




Product is mentioned by name in Cleaning & Care guide and is approved for use; if relevant, dilution is stated

Active ingredient/level is listed in Cleaning & Care guide as approved for use; if relevant, dilution is stated

# Clorox Healthcare® Equipment Compatibility Guide

		Bleach				Hydrogen Peroxide	Quat	
<b>Furniture/Apparel</b>								
		Clorox Healthcare® Bleach Germicidal Wipes	Clorox Healthcare® Bleach Germicidal Cleaners	Clorox Healthcare® Fuzion Cleaner Disinfectant	Dispatch® Hospital Cleaner Disinfectant Towels with Bleach	Clorox® Germicidal Bleach	Clorox Healthcare® Hydrogen Peroxide Cleaner Disinfectant Wipes	Clorox Healthcare® VersaSure Alcohol-Free Disinfectant Wipes
Manufacturer	Product							
Kwalu	All furniture products							
healthCentric	IC+ Upholstery Solution™							
Prime Medical	SAF-T™ Scrubs							
Midmark	Midmark® + Ritter® Chairs, Exam Tables + Seating (Upholstery)					1:10		
	Synthesis® Cabinetry + Midmark® Workstations					1:10		
DentalEZ Equipment Solutions	JV Generation Chair					1:10		
Graham-Field	LUMEX FR577RG/RGH Series Clinical Care Recliners					1:10		
Clinton Industries	Lab-X Series Padded Blood Drawing Chairs							
Activeaid, Inc.	Model 285/20 Tilt-In-Space Adult Shower/Commode/Wheel Chair							
Harloff Company, Inc.	Powder Coated and Stainless Steel Carts					0.05%	Clorox Disinfecting Wipes	
Wheelchairs of Kansas	Model 38060000 Bariatric Lift and Transfer Unit							
A-Dec	Dental Chair Upholstery							

**Legend**

-  Product is mentioned by name in Cleaning & Care guide and is approved for use; if relevant, dilution is stated
-  Active ingredient/level is listed in Cleaning & Care guide as approved for use; if relevant, dilution is stated








# Clorox Healthcare® Equipment Compatibility Guide

		Bleach				Hydrogen Peroxide	Quat
<b>Ultrasound/Transducer Probes (GE)</b>							
		Clorox Healthcare® Bleach Germicidal Wipes	Clorox Healthcare® Bleach Germicidal Cleaners	Clorox Healthcare® Fuzion Cleaner Disinfectant	Dispatch® Hospital Cleaner Disinfectant Towels with Bleach	Clorox® Germicidal Bleach	Clorox Healthcare® Hydrogen Peroxide Cleaner Disinfectant Wipes
Manufacturer	Product						
General Electric	<b>LOGIQ™ TRANSDUCERS/ ULTRASOUNDS</b>						
	10L, 10S						
	10LB, 10LB-RS						
	10S-RS						
	11L, 11L-D						
	12L						
	12L-RS, 12L-SC						
	12S-D						
	12S-RS						
	16-RS						
	16L-RS						
	3.5C, 3.5CS, 3.5 gray lens, 3C, 3Cb, 3CRF, 3CRF-D, 3C-RS, 3S, 3SC-RS, 3S-D, 3S-LC, 3SR, 3S-RC, 3S-RS, 3S-SC,						
	3V, 3V-D						
	4C						
	4C-A, 4C-D						
	4C-RS, 4C-SC						
	4D10L						
	4D16L, 4D3C-L, 4D5C-L, 4DE7C, 4S, 4V-D						
	4Vc-D						
	4D8C						
	546L-V						
	5CS						
	5C, 5S, 5S-LC,						
5S-RS							
6S, 6S-D							
6S-RS							
739L-V							
7L, 7S, 7S-R5							
8C, 8C-RS							
8L							
8L-RS							



**Legend**

- Product is mentioned by name in Cleaning & Care guide and is approved for use; if relevant, dilution is stated
- Active ingredient/level is listed in Cleaning & Care guide as approved for use; if relevant, dilution is stated








# Clorox Healthcare® Equipment Compatibility Guide

		Bleach				Hydrogen Peroxide	Quat	
<h2>Ultrasound/Transducer Probes (GE)</h2>								
		Clorox Healthcare® Bleach Germicidal Wipes	Clorox Healthcare® Bleach Germicidal Cleaners	Clorox Healthcare® Fuzion Cleaner Disinfectant	Dispatch® Hospital Cleaner Disinfectant Towels with Bleach	Clorox® Germicidal Bleach	Clorox Healthcare® Hydrogen Peroxide Cleaner Disinfectant Wipes	Clorox Healthcare® VersaSure Alcohol-Free Disinfectant Wipes
Manufacturer	Product							
General Electric	9L, 9L-D, 9L-RS							
	AB2-5, AB2-7, AB2-7-D, AB2-7-RS, AB4-8							
	AC2-5							
	BE9C, BE9CS, BE9CS-D, BE9CS-RS							
	C1-5-D							
	C1-5-RS, C1-6-D, C1-6VN-D							
	C31, C3-10-D, C55, C551, CAE, CS							
	C2-5-RC, C2-9-D, C2-9VN-D							
	C3-10-D, C4-8-D							
	C358, C36, C364, C386, C721, CFB, CZB							
	Dual Probe Vscan							
	ERB							
	E72, E721							
	E8C, E8C-RS, E8CS, E8CS-RC, E8CS-RS, E8CS-SC							
	eM6C							
	eM6C G2							
	G3C Probe on Vscan, C3S Probe on Vscan							
	i12L, i12L-RS							
	IC5-9, IC5-9W-RS							
	I739, I739-RS							
	IC5-9-D, IC5-9H, IC9-RS							
	L39, L76, L764, LB, LD, LE, LH, LI, LT							
	LA39							
	L12n-SC, L3-12-D, L4-12t-RS, L6-12-RC, L6-12-RS, L8-18i-D, L8-18i-RS, L8-18i-SC							
	L12n-RS							
	M12L, M12L-H, M3S, M4S, M4S-D, M4S-RS, M5Sc-D, M5Sc-RS, M6C, M7C, M7C-H, ML6-15, ML6-15-D, ML6-15-RS,							

**Legend**

-  Product is mentioned by name in Cleaning & Care guide and is approved for use; if relevant, dilution is stated
-  Active ingredient/level is listed in Cleaning & Care guide as approved for use; if relevant, dilution is stated

# Clorox Healthcare® Equipment Compatibility Guide








		Bleach				Hydrogen Peroxide	Quat	
<b>Ultrasound/Transducer Probes (GE)</b>								
		Clorox Healthcare® Bleach Germicidal Wipes	Clorox Healthcare® Bleach Germicidal Cleaners	Clorox Healthcare® Fuzion Cleaner Disinfectant	Dispatch® Hospital Cleaner Disinfectant Towels with Bleach	Clorox® Germicidal Bleach	Clorox Healthcare® Hydrogen Peroxide Cleaner Disinfectant Wipes	Clorox Healthcare® VersaSure Alcohol-Free Disinfectant Wipes
Manufacturer	Product							
General Electric	MTZ, MZ							
	M3-8							
	PA2-5P							
	PCW4.0							
	RAB2-5, RAB2-5-D, RAB2-5-RS, RAB2-5L, RAB-2-6-RS, RAB4-8-D, RAB4-8-RS, RAB4-8L, RAB4-8P, RAB6-D, RAB6-RS, RIC5-9, RIC5-9-D, RIC5-9-RS, RIC5-9A-RS, RIC5-9H, RIC5-9W, RIC5-9W-RS, RIC6-12-D, RRE5-10-D, RSP5-12, RSP6-12, RSP6-16, RSP6-16-D, RSP6-16-RS							
	RNA5-9, RNA5-9-D, RNA5-9-RS							
	RAM3-8, RM14L, RM6C, RRE6-10, RRE6-10-D, RSM5-14,							
	S222, S317, S611							
	S220, S316, SCW2.0							
	SP10-16-RS, SP4-10, SP6-12							
	T739, T739-RS							
	Vscan Access convex, Vscan dual (sector/linear)							
	Vscan sector							
	LogiQ S8					1:10 of 5%	Wipes	

**Legend**





Product is mentioned by name in Cleaning & Care guide and is approved for use; if relevant, dilution is stated  
 Active ingredient/level is listed in Cleaning & Care guide as approved for use; if relevant, dilution is stated

# Clorox Healthcare® Equipment Compatibility Guide

		Bleach				Hydrogen Peroxide	Quat	
<b>Ultrasound/Transducer Probes (Phillips)</b>		 Clorox Healthcare® Bleach Germicidal Wipes	 Clorox Healthcare® Bleach Germicidal Cleaners	 Clorox Healthcare® Fuzion Cleaner Disinfectant	 Dispatch® Hospital Cleaner Disinfectant Towels with Bleach	 Clorox® Germicidal Bleach	 Clorox Healthcare® Hydrogen Peroxide Cleaner Disinfectant Wipes	 Clorox Healthcare® VersaSure Alcohol-Free Disinfectant Wipes
Manufacturer	Product							
Phillips	ULTRASOUND TRANSDUCERS & PROBES (SYSTEM NAMES):							
	3D6-2 (HD11), 3D8-4, 3D9-3v (HD11, HD15, iU22) ,	T/C	T/C	T/C		T/C		
	7.5MHZ Endo	T/C	T/C	T/C		T/C		
	C5-1 (CX50, HD15, iE33, iU22), C5-2 (HD5, HD6, HD7, HD11, HD15, iE33, iU22) , C6-2 Compact (CX30, Sparq), C6-3 , C8-4v, C8-5 , C8-5 Compact (CX30, CX50) ,	T/C	T/C	T/C		T/C		
	C9-3io , C9-4, C9-4ec, C9-5ec	T/C	T/C	T/C		T/C		
	L8-4, L9-3 , L11-3, L12-3 (HD5, HD6, HD7, HD11, HD15), L12-4 Compact (CX30), L12-5 50 Compact (CX50), L12-5 50 mm, L15-7io (HD11, HD15, iE33, iU22), L17-5	T/C	T/C	T/C		T/C		
	S3-1, S4-1 (iU22), S4-2, S4-2 Compact (CX30, Sparq), S5-1 (CX50, iE33, iU22), S5-2, S8, S8-3, S8-3 Compact (CX30, CX50, HD5), S12, S12-4, S12-4 Compact (CX50)	T/C	T/C	T/C		T/C		
	V6-2 (HD11XE, HD15, iU22), V8-4	T/C	T/C	T/C		T/C		
	X3-1, X5-1, X7-2 (iE33, iU22)	T/C	T/C	T/C		T/C		
	3D6-2 (iU22), 3D9-3v (Affiniti, ClearVue, EPIQ)	T/C	T/C	T/C		T/C		
	BP10-5ec	T/C	T/C	T/C		T/C		
	C5-1 (Affiniti, EPIQ), C5-2 - ClearVue, C6-2 Compact (Affiniti), C8-4v, C8-5 Compact (Affiniti, EPIQ),	T/C	T/C	T/C		T/C		








**Legend**

 Product is mentioned by name in Cleaning & Care guide and is approved for use; if relevant, dilution is stated  
 Active ingredient/level is listed in Cleaning & Care guide as approved for use; if relevant, dilution is stated

- C = Cable, connector, and strain reliefs
- H = Handle and steering mechanism
- T = Transducer and flexible shaft




# Clorox Healthcare® Equipment Compatibility Guide

		Bleach				Hydrogen Peroxide	Quat	
<h2>Ultrasound/Transducer Probes (Phillips)</h2>								
		Clorox Healthcare® Bleach Germicidal Wipes	Clorox Healthcare® Bleach Germicidal Cleaners	Clorox Healthcare® Fuzion Cleaner Disinfectant	Dispatch® Hospital Cleaner Disinfectant Towels with Bleach	Clorox® Germicidal Bleach	Clorox Healthcare® Hydrogen Peroxide Cleaner Disinfectant Wipes	Clorox Healthcare® VersaSure Alcohol-Free Disinfectant Wipes
Manufacturer	Product							
Phillips	C9-2, C9-3v, C9-4v, C9-4v Compact, C10-3v, C10-4ec		T/C	T/C	T/C		T/C	
	L12-3 (Affiniti, CX50, EPIQ), L12-4 X7-2 (EPIQ), L12-4 Compact (Affiniti, EPIQ), L15-7io (Affiniti, CX30, CX50, EPIQ), L15-7io (HD11, HD15, IE33, iU22), L18-5		T/C	T/C	T/C		T/C	
	S4-1 (ClearVue), S4-2 Compact (Affiniti), S5-1 (Affiniti, EPIQ), S8-3 Compact (Affiniti, EPIQ), S12-4 Compact (Affiniti, EPIQ)		T/C	T/C	T/C		T/C	
	V6-2 (Affiniti, ClearVue, EPIQ, HD5), VL13-5, VL13-5 Compact		T/C	T/C	T/C		T/C	
	X5-1 (EPIQ), X6-1		T/C	T/C	T/C		T/C	
	1.9MHz CW Pencil, 1.9MHz TCD		T/C	T/C	T/C		T/C	
	5.0MHz CW Pencil		T/C	T/C	T/C		T/C	
	D2cwc, D2tcd, D5cwc		T/C	T/C	T/C		T/C	
	TCD		T/C	T/C	T/C		T/C	
	S6-2mpt S7-3t, S7-2t Omni, S8-3t		C	C	C		T/C/H	
	OMNI III TEE		C	C	C		T/C/H	
	X7-2t		C	C	C		T/C/H	
	L10-4lap		T/C	T/C	T/C		T/C/H	
	15-6L		T/C	T/C	T/C		C	
	CO2 Transducer	M1460A						
	Avalon Fetal Transducer							

**Legend**

 Product is mentioned by name in Cleaning & Care guide and is approved for use; if relevant, dilution is stated








 Active ingredient/level is listed in Cleaning & Care guide as approved for use; if relevant, dilution is stated

**C** = Cable, connector, and strain reliefs



**H** = Handle and steering mechanism

**T** = Transducer and flexible shaft

# Clorox Healthcare® Equipment Compatibility Guide

		Bleach				Hydrogen Peroxide	Quat	
<b>Ultrasound/Transducer Probes (Other)</b>								
		Clorox Healthcare® Bleach Germicidal Wipes	Clorox Healthcare® Bleach Germicidal Cleaners	Clorox Healthcare® Fuzion Cleaner Disinfectant	Dispatch® Hospital Cleaner Disinfectant Towels with Bleach	Clorox® Germicidal Bleach	Clorox Healthcare® Hydrogen Peroxide Cleaner Disinfectant Wipes	Clorox Healthcare® VersaSure Alcohol-Free Disinfectant Wipes
Manufacturer	Product							
Siemens	ULTRASOUND TRANSDUCERS & PROBES (SYSTEM NAMES):							
	Acuson CV70 , Acuson SC2000 , Acuson Cypress , Acuson X500 , Acuson X300, Acuson Antares , Acuson X150, and Sonoline G20 UltraSound Systems							
	3V2c, 4V1c, 5.0L45, 5.0P10, 6C1HD, 7.5L75S, 8V3, 10V4							
	BE9-4							
	C6-2, C7F2, C8-5, CW2, CW5, C5-2, C5F1, CH4-1, CH6-2, CX5-2							
	EC9-4, EV9-4							
	L10-5							
	MPT7-4							
	P4-2, P9-4, P10-4, PH4-1, PX4-1							
	AUX CW							
	VF13-5-SP, VF7-3, VFX13-5, V5Ms, VF13-5-SP, V7M							
FUJIFILM SONOSITE, INC. www.sonosite.com/support/cleaners-disinfectants	SYSTEMS							
	Edge™							
	Edge II							
	FC1							
	SII							
X-Porte								

**Legend**

-  Product is mentioned by name in Cleaning & Care guide and is approved for use; if relevant, dilution is stated
-  Active ingredient/level is listed in Cleaning & Care guide as approved for use; if relevant, dilution is stated








# Clorox Healthcare® Equipment Compatibility Guide

		Bleach				Hydrogen Peroxide	Quat	
<b>Ultrasound/Transducer Probes (Other)</b>								
		Clorox Healthcare® Bleach Germicidal Wipes	Clorox Healthcare® Bleach Germicidal Cleaners	Clorox Healthcare® Fuzion Cleaner Disinfectant	Dispatch® Hospital Cleaner Disinfectant Towels with Bleach	Clorox® Germicidal Bleach	Clorox Healthcare® Hydrogen Peroxide Cleaner Disinfectant Wipes	Clorox Healthcare® VersaSure Alcohol-Free Disinfectant Wipes
Manufacturer	Product							
FUJIFILM SONOSITE, INC. www.sonosite.com/support/cleaners-disinfectants	TRANSDUCERS							
	C11n, C11x, C11xp, C60n, C60x, C60xf, C60xi, C60xp, C8x, D2xp, HFL38x, HFL38xi, HFL38xp, HFL50x, HFL50xp, HSL25x, HSL25xp, ICTn, ICTx, ICTxp, L25n, L25x, L25xp, L38n, L38v, L38x, L38xi, L38xp, L52n, L52x, L52x Armored, P10x, P10xp, P21n, P21x, P21xp, rC60xi, rP19x, rP19xp, SLAx							
	C60v, L25v, P21v,							
	ACCESSORIES							
	Battery Power Pack, External Power Supply, Power Supply, Power Pack, TTC-Legacy, X-Porte Stand							
	Dock, FC1 Stand, H-Universal Stand, SII Stand, TTC-Edge, TTC_S Series, V-Universal Stand							
	ECG Cable, Mini Dock, Edge II Stand, Edge Stand							
Welch Allyn	SureTemp® Therms and Probes							
Toshiba	Transducers							
	Diagnostic Ultrasound System Aplio i900, i800 and i700							
Echosens	FibroScan 502/530 Touch Ultrasound and Probes							

**Legend**

- Product is mentioned by name in Cleaning & Care guide and is approved for use; if relevant, dilution is stated
- Active ingredient/level is listed in Cleaning & Care guide as approved for use; if relevant, dilution is stated

# Clorox Healthcare® Equipment Compatibility Guide








		Bleach				Hydrogen Peroxide	Quat	
<b>Keyboards/Phones/ Cases</b>								
		Clorox Healthcare® Bleach Germicidal Wipes	Clorox Healthcare® Bleach Germicidal Cleaners	Clorox Healthcare® Fuzion Cleaner Disinfectant	Dispatch® Hospital Cleaner Disinfectant Towels with Bleach	Clorox® Germicidal Bleach	Clorox Healthcare® Hydrogen Peroxide Cleaner Disinfectant Wipes	Clorox Healthcare® VersaSure Alcohol-Free Disinfectant Wipes
Manufacturer	Product							
Whitney Medical Solutions	eShield® Screen Protectors and Cases							
KeySource International	San-a-Key® Keyboards							
Man and Machine	Keyboards and Mice							
Otterbox	Phone and iPad® cases							
Seal Shield	Medical Grade Keyboards, Mice, and Screen Protectors							

**Legend**



Product is mentioned by name in Cleaning & Care guide and is approved for use; if relevant, dilution is stated  
 Active ingredient/level is listed in Cleaning & Care guide as approved for use; if relevant, dilution is stated

# Clorox Healthcare® Equipment Compatibility Guide

		Bleach				Hydrogen Peroxide	Quat	
<b>Lights</b>								
		Clorox Healthcare® Bleach Germicidal Wipes	Clorox Healthcare® Bleach Germicidal Cleaners	Clorox Healthcare® Fuzion Cleaner Disinfectant	Dispatch® Hospital Cleaner Disinfectant Towels with Bleach	Clorox® Germicidal Bleach	Clorox Healthcare® Hydrogen Peroxide Cleaner Disinfectant Wipes	Clorox Healthcare® VersaSure Alcohol-Free Disinfectant Wipes
Manufacturer	Product							
Skytron	Precision HD Handle Camera System							
	OR Lights							
Welch Allyn	Exam Lighting							
	Green Series™ 300 General Exam Lights							
	ProXenon 350 Surgical Illuminator					1:10		
Trumpf	iLED 5/TruLight 5000 Lighting System							
Sunoptic Technologies	Surgical S400T Titan 400HP/TITAN 180 Xenon Illuminator							

**Legend**



Product is mentioned by name in Cleaning & Care guide and is approved for use; if relevant, dilution is stated  
 Active ingredient/level is listed in Cleaning & Care guide as approved for use; if relevant, dilution is stated








# Clorox Healthcare® Equipment Compatibility Guide

		Bleach				Hydrogen Peroxide	Quat	
<h2>Vital Sign Monitors/ Consoles</h2>								
		Clorox Healthcare® Bleach Germicidal Wipes	Clorox Healthcare® Bleach Germicidal Cleaners	Clorox Healthcare® Fuzion Cleaner Disinfectant	Dispatch® Hospital Cleaner Disinfectant Towels with Bleach	Clorox® Germicidal Bleach	Clorox Healthcare® Hydrogen Peroxide Cleaner Disinfectant Wipes	Clorox Healthcare® VersaSure Alcohol-Free Disinfectant Wipes
Manufacturer	Product							
Barco	Eonis® Diagnostic Displays							
GE Healthcare	Transport Pro							
	ApexPro Telemetry System				1:10-1:100			
Midmark	ApexPro Antenna Infrastructure, Transmitter and Receiver				1:10-1:100			
	IQvitals							
	IQvitals® Blood Pressure Cuffs							
Phillips	IntelliVue Patient Mx400/450/500/600/700/800							
	IntelliVue Telemetry Transmitters							
	IntelliVue Monitors							
Welch Allyn	Avalon Fetal Monitors							
	Connex Vital Signs							
	Connex Integrated Wall System				1:10			
	Accessory Cable Management Mobile Stand 4800/4900 Series							
	CareTemp Touch Free Thermometer				1:10			
	Propaq Encore® Blood Pressure Cuffs							
	Harvey™ Elite® Stethoscopes							
	LumiView™ Binocular Microscope							
MindRay	Digital MacroView™ Otoposcopes and Halogen HPX® Illuminators							
	Accutorr 7 Vital Signs Monitor							
	Passport 12/Passport 8 Patient Monitor							
Mortara Instrument, Inc.	ELI 280/150c/250c 12-lead Resting Electrocardiograph				1:500 to 1:10			
Capsule Tech	SmartLinx Vitals Plus				10%			
Exergen	TAT5000 Temporal Thermometer							

**Legend**

- Product is mentioned by name in Cleaning & Care guide and is approved for use; if relevant, dilution is stated
- Active ingredient/level is listed in Cleaning & Care guide as approved for use; if relevant, dilution is stated

# Clorox Healthcare® Equipment Compatibility Guide

		Bleach				Hydrogen Peroxide	Quat	
<b>Radiology</b>								
		Clorox Healthcare® Bleach Germicidal Wipes	Clorox Healthcare® Bleach Germicidal Cleaners	Clorox Healthcare® Fuzion Cleaner Disinfectant	Dispatch® Hospital Cleaner Disinfectant Towels with Bleach	Clorox® Germicidal Bleach	Clorox Healthcare® Hydrogen Peroxide Cleaner Disinfectant Wipes	Clorox Healthcare® VersaSure Alcohol-Free Disinfectant Wipes
Manufacturer	Product							
AGFA Healthcare	Computed Radiology Cassettes							
	DR Detectors (Dx-D 10C, 10G, 20C, 20G)							
	DX-D 100 Digital Radiography X-Ray System							
Hologic	GemStar® Infusion Pumps							
InFab	Aprons and Gloves							
GE Healthcare	Optima XR220amx X-Ray System					5250-6150 ppm		
	Senographe Pristina Mammography System							
Konica Minolta	CS-1 Regius							

**Legend**



Product is mentioned by name in Cleaning & Care guide and is approved for use; if relevant, dilution is stated  
 Active ingredient/level is listed in Cleaning & Care guide as approved for use; if relevant, dilution is stated

# Clorox Healthcare® Equipment Compatibility Guide

		Bleach				Hydrogen Peroxide	Quat	
<b>Ventilator Systems</b>								
		Clorox Healthcare® Bleach Germicidal Wipes	Clorox Healthcare® Bleach Germicidal Cleaners	Clorox Healthcare® Fuzion Cleaner Disinfectant	Dispatch® Hospital Cleaner Disinfectant Towels with Bleach	Clorox® Germicidal Bleach	Clorox Healthcare® Hydrogen Peroxide Cleaner Disinfectant Wipes	Clorox Healthcare® VersaSure Alcohol-Free Disinfectant Wipes
Manufacturer	Product							
Covidien	Puritan Bennett							
	540 series (409 was OK'd for touchscreen and arm)							
	840 series (409 was OK'd for touchscreen and arm)							
	980 series (409 was OK'd for touchscreen and arm)							
Kendall SCD	700 Sequential Compression System							
AV Impluse	6060 Controller and Accessories							
Phillips	Triology 100 System							
	Respironics V60/Plus Ventilato					5%; 1:10		
Getinge-Maquet	SERVO i/s Ventilator System and Tube							

**Legend**

- Product is mentioned by name in Cleaning & Care guide and is approved for use; if relevant, dilution is stated
- Active ingredient/level is listed in Cleaning & Care guide as approved for use; if relevant, dilution is stated



# Clorox Healthcare® Equipment Compatibility Guide

		Bleach				Hydrogen Peroxide	Quat	
<b>Healthcare IT</b>								
		Clorox Healthcare® Bleach Germicidal Wipes	Clorox Healthcare® Bleach Germicidal Cleaners	Clorox Healthcare® Fuzion Cleaner Disinfectant	Dispatch® Hospital Cleaner Disinfectant Towels with Bleach	Clorox® Germicidal Bleach	Clorox Healthcare® Hydrogen Peroxide Cleaner Disinfectant Wipes	Clorox Healthcare® VersaSure Alcohol-Free Disinfectant Wipes
Manufacturer	Product							
Enovate Medical	Computer Carts (EMC)							
RICOH USA	Interactive Whiteboard							
Panasonic	TOUGHBOOK® 20, 33, 55, A3, G1, L1, N1, T1							
GETT	Cleankeys® (wired and wireless), InduSense®							
Onyx Healthcare	Mobile Cart Computing Solution Venus: 243/223/222/192							
	Fanless Slim HIT Panel PC Solution ONYX-BE182/1723/1731/1531/1222/1022							
	Patient Monitor & Smart View Medical Solution ZEUS- 248/228/198							
	Medical All-In-One Solution AIO-MATE-2403/2203/1903							
	Healthcare Infotainment Solution ONYX-BE321/BE182/1531/1222 MedPC- 2100/1600							
	Mobile Medical Assistant MD116/MD101							
	Medical Display Series Solution MEDDP- 524/522/519/518/517/515							
HP	Healthcare Edition Notebook/ All in One							
Stanley Healthcare	Hugs and Kisses Tags							
GE Healthcare	Carescape Central Station MPI00					5.25%; 1:10 dilution		
TouchPoint InterMetro Industries Corp.	Metro 1800 Series Mobile Workstation							
Capsule Tech	SmartLinx Axon					10%	Clorox Disinfecting Wipes	
	SmartLinx Device Identification Module					10%		
Howard Medical	E Series Computerized Work Station							
SealShield	Keyboards, mouse products, screen protectors					1/2 cup to 1 gallon		

**Legend**

- Product is mentioned by name in Cleaning & Care guide and is approved for use; if relevant, dilution is stated
- Active ingredient/level is listed in Cleaning & Care guide as approved for use; if relevant, dilution is stated

# Clorox Healthcare® Equipment Compatibility Guide

		Bleach					Hydrogen Peroxide	Quat
<b>Other</b>								
		Clorox Healthcare® Bleach Germicidal Wipes	Clorox Healthcare® Bleach Germicidal Cleaners	Clorox Healthcare® Fuzion Cleaner Disinfectant	Dispatch® Hospital Cleaner Disinfectant Towels with Bleach	Clorox® Germicidal Bleach	Clorox Healthcare® Hydrogen Peroxide Cleaner Disinfectant Wipes	Clorox Healthcare® VersaSure Alcohol-Free Disinfectant Wipes
Manufacturer	Product							
Blickman	Surgical Steel							
Carefusion	Pyxis® Supply Systems							
Roche	Cobas® Urisys 1100® Urine Analyzers							
Sechrist	Chambers							
RF Surgical Systems	RF Assure Detection System							
Medline	BioCon 750 Bladder Scanner							
PhysioControl	LIFEPAK 15 Monitor/Defibrillator							
WelchAllyn	Suresight® Vision Screener and Autorefractor					1:10		
GE Healthcare	Giraffe OmniBed Carestation CS1					0.50%		
Custom Ultrasonics	83-S6 Preprocessing Sink					1:32		
AirClean Systems	AC600 Series Laminar Flow Workstation					1:10		
Gentec	882VR Continuous Suction Regulators							
Tomey USA	Pachymeter Probe							
LivaNova Sorin Group	T Heater-Cooler System							
Dwyer Precision Products	BreathCall Series Mounting							Broad Spectrum
Chart Industries	AirSep QuietLife 5 EliteOxygen Concentrator							
AirSep	NewLife EliteOxygen Concentrator							
Germfree	BVBI Series/BZ Series Vertical Laminar Flow Workstation					5%		
	VersaFlow Compounding Aseptic Isolators					<1000 ppm		
Besmax Healthcare, Inc.	Curaplex Oxygen Nipple Connector					0.5%		
Zimmer Biomet Dornoch Medical Systems, Inc.	UL-EV100 Transposal Ultra Evacuation Unit					<1200 ppm		
Covaris	cryoPREP Extraction System					10%		
Fresenius Medical Care	2008K series							
	2008T series							
	Granuflo Concentrate Unit 1							
Draeger	Perseus A500 series							
Nanosonics	Trophon - chamber, exterior							

**Legend**

- Product is mentioned by name in Cleaning & Care guide and is approved for use; if relevant, dilution is stated
- Active ingredient/level is listed in Cleaning & Care guide as approved for use; if relevant, dilution is stated